

## WHAT'S INCLUDED IN YOUR ALPHA-STIM® KIT?

### ALPHA-STIM® M

The Alpha-Stim M comes complete and ready to use with: (1) Earclip Electrodes set; (1) 50 mL bottle of Alpha-Stim® Conducting Solution with (1) empty bottle for use with Alpha-Stim® Conducting Solution; 2 Leadwires sets; (4) Silver AS-Trode single person multiple use self-adhesive electrodes; 2 Smart Probes; 100 Probe Electrode Pads (PEPS™); 256 Earclip Electrode Pads (EEPS™); Owner's Manual; Lanyard; Storage case; (2) AA 1.5 volt lithium batteries.



### ALPHA-STIM® AID

The Alpha-Stim AID comes complete and ready to use with: (1) Earclip Electrodes set; (1) 50 mL bottle of Alpha-Stim® Conducting Solution with (1) empty bottle for use with Alpha-Stim® Conducting Solution; 256 Earclip Electrode Pads (EEPS™); Owner's Manual; Lanyard; Storage case; (2) AAA 1.5 volt lithium batteries.



## PRESCRIBING INFORMATION

### EPI is ISO Certified

Electromedical Products International, Inc. is an International Standards Organization (ISO) certified establishment. ISO is an International organization working with some 140 countries and the United Nations to maintain standards for all applications of technology for global industry. Requirements for the medical device industry relate to design controls, risk management, environmental controls, special processes (e.g. software validation), traceability, record retention, and regulatory actions such as vigilance.

### Electromagnetic Interference

This equipment has been independently tested by outside agencies and found to comply with the limits of Comité International Spécial des Perturbations Radioélectriques (CISPR). These limits are designed to provide reasonable protection against harmful interferences in a residential or clinical environment. However, it is still possible that interference could occur in a particular environment. In case interference does occur, increase the distance between this device and the equipment it interferes with. Consult Electromedical Products International, Inc. if the problem persists.

### Indications

The Alpha-Stim M is a precision medical device and is an effective treatment with broad applications for a variety of syndromes involving pain, and for the management of anxiety and insomnia, or for the short-term relief of symptoms associated with these indications. The Alpha-Stim AID is a precision medical device used for cranial electrotherapy stimulation for the treatment of anxiety and insomnia.

### Warnings and Precautions

- Safety of stimulation has not been established during pregnancy.
- Do not stimulate directly on the eyes or press the probes over the carotid sinus (on the neck near the larynx). Application of the electrodes near the thorax may increase the risk of cardiac fibrillation.
- Potential hazard from simultaneous connection of a patient to a high frequency surgical medical equipment and stimulator may result in burns and possible damage to the stimulator.
- Operation in close proximity (e.g. 1m) to a shortwave or microwave therapy medical equipment may produce instability in the stimulator output.
- Modification of your Alpha-Stim device or accessories is not allowed, it could result in injury.
- Do not place Alpha-Stim in direct contact with lint, dust, light (including sunlight).
- For external use only. Small parts, keep out of reach of children, avoid inhalation or swallowing. Do not allow children to use or handle this device.
- This device is intended to be used in individuals who are 18 years of age or older.
- Brain development continues beyond 18 years of age. The long-term effects of electrical stimulation from this device on the brain has not been studied.
- Do not operate potentially dangerous machinery or vehicles during treatment, and in some cases for several hours after treatment.
- Do not plug leadwires into wall sockets or line cord receptacles under any circumstances. Doing so could result in severe shock or burns whether the leadwires are attached to the stimulator or not.
- Patients should be monitored by their physician for signs of worsening.
- If a patient experiences headaches while using the device, they should consult their physician prior to continued use of the device.
- Caution Statement for United States: Federal law (USA only) restricts this device to sale by, or on the order of a licensed healthcare practitioner. Outside the USA it is available worldwide without a prescription but consultation with a qualified healthcare professional is recommended for difficult and unresponsive problems or when used with pharmaceuticals or other therapeutic intervention.

### Contraindications

Use of an Alpha-Stim device is contraindicated with implanted pacemakers or implanted or wearable defibrillators.

### Adverse Effects

Adverse effects are usually rare (occurring less than 1% of the time), mild, and self-limiting.

- dizziness
- skin irritation/electrode burns
- headaches



### Electromedical Products International, Inc.

2201 Garrett Morris Parkway  
Mineral Wells, TX • 76067-9034 • USA  
TEL: 1-800-367-7246 in the USA or +1 940-328-0788  
FAX: 940-328-0888 • EMAIL: alpha-stim@epii.com  
[www.alpha-stim.com](http://www.alpha-stim.com)

Made in the USA and in China to the exacting specifications of EPI, USA.  
© 2025 Electromedical Products International, Inc. All rights reserved.  
Alpha-Stim and the Alpha-Stim logo are registered trademarks of  
Electromedical Products International, Inc. Manufactured under US Patents  
8,612,008; 8,457,765; and 8,463,406.

# TAKE CHARGE OF YOUR LIFE with ALPHA-STIM®



THE DRUG-FREE, CLINICALLY PROVEN TREATMENT FOR  
**ANXIETY, INSOMNIA, AND PAIN**

ALPHA-STIM®



MKT-CB-001 VER. 1

# Are you tired of living with anxiety, insomnia or pain? **YOUR QUEST FOR RELIEF ENDS HERE.**

Alpha-Stim is a non-invasive, non-drug medical device that is FDA cleared to safely and effectively treat anxiety, insomnia, and pain. It can be used in the comfort of your own home while doing normal, everyday activities.



“

Linda S.

## HEALTHY SLEEP

My doctor prescribed Alpha-Stim AID for my chronic insomnia. The first night I tried it, I went to sleep without any medication for the first time in years.

”



“

Laureen W.

## EFFECTIVE RELIEF

From the very first time I used an Alpha-Stim AID in my therapist's office, I experienced relief from crippling anxiety. I'd been searching for relief from anxiety for a few years, and no other drug or therapy has been as effective as Alpha-Stim.

”



“

Justin P.

## CHANGED MY LIFE

The Alpha-Stim M has changed my life. After a decade of debilitating chronic pain, I can walk, sleep and function again. No more prescription drug stupor and spiraling negative side effects.

”

### USE ALPHA-STIM® M FOR:



- Pain
- Anxiety
- Insomnia

### USE ALPHA-STIM® AID FOR:



- Anxiety
- Insomnia

### WHY ALPHA-STIM®

- Non-drug, FDA cleared medical device
- Easy-to-use, portable, and painless
- Fast acting, long-lasting, cumulative relief
- No risk of addiction and no lasting side effects

### WHAT TO EXPECT

Alpha-Stim delivers fast, long-lasting and cumulative relief, with many users experiencing relief after their first treatment. Use daily until symptom reduction is noticeable. Then maintain your results with 2-3 treatments a week, or as needed.

## SPEAKING THE LANGUAGE OF THE BRAIN & BODY

Psychiatric disorders, such as anxiety and insomnia, are associated with alterations in electric activity in the brain. Alpha-Stim CES is a form of non-invasive brain stimulation that applies a safe, patented low amplitude (mild) electrical waveform to the brain via Earclip Electrodes providing rapid, effective, and lasting symptom relief.

Pain transmits through your body as a sensation along billions of nerve cells designated to transmit pain messages through electrochemical signals. Alpha-Stim MET is the process of directing electrical microcurrent via Smart Probes or self-adhesive AS-Trodes™ at the site of your pain.

Over 100 independent clinical studies have proven the safety and effectiveness of Alpha-Stim.

## ARE YOU READY TO TAKE BACK YOUR LIFE?

Try Alpha-Stim today with a 30-day money-back satisfaction guarantee.

Scan QR code below to get started at [Alpha-Stim.com](http://Alpha-Stim.com) or call 1-800-FOR-PAIN.

